



Donning & Doffing of Reusable Face Masks

Developed by:	Approved by:	Date created:	Last revision:
Jason M. Moffatt	G. Kaluzniak	May 2020	December 1, 2022
Possible Hazards Present	Personal Protection Required	Additional Training	
<ul style="list-style-type: none"> - Biohazards - Dusts - Mists 	<ul style="list-style-type: none"> - Reusable face mask - Site requirements including hard hat, glasses, gloves. - 	<ul style="list-style-type: none"> -Orientation to reusable masks. -J&G Group of Companies, COVID Hazard Assessment. 	

Safe Work Procedure

Donning Procedure:

1. Perform hand hygiene with alcohol-based hand sanitizer or a thorough hand washing before you remove any personal eyewear or safety glasses.
2. Perform hand hygiene with alcohol-based hand sanitizer or a thorough hand washing again prior to donning the face mask. Ensure you are maintaining a safe distance of at least 2m (6 feet) from coworkers.
3. Donn the face mask and secure it to your face, over your mouth and nose as designed by the manufacturer.
4. Donn personal eyewear or safety glasses after the face mask is comfortable.
5. Perform hand hygiene with alcohol-based hand sanitizer or a thorough hand washing anytime you touch the mask while wearing.

Doffing Procedure:

1. Perform hand hygiene with alcohol-based hand sanitizer or a thorough hand washing before starting the doffing procedure.
2. Remove personal eyewear or safety glasses prior to removing the face mask.
3. Doff the face mask with the eyes closed. If the mask is reusable, place it in a bag. If the mask is disposable, discard it in an approved waste container.
4. Perform hand hygiene with alcohol-based hand sanitizer or a thorough hand washing.
5. Donn personal eyewear or safety glasses again if required.
6. Perform hand hygiene with alcohol-based hand sanitizer or a thorough hand washing.



<p>Documentation/Legislation</p> <ul style="list-style-type: none">- WPSH Reg. MR. 217/2006- Shared Health Emergency Response Services- Safe Work Procedure “Donning and Doffing of Reusable Face Masks During COFID-19 Pandemic”- CDC Guidelines: https://www.cdc.gov/coronavirus/2019-ncov/hcp/guidance-for-ems.html	<p>This Safe Work Procedure will be reviewed anytime the task, equipment or materials change and at a minimum every three years.</p>
---	---